

# LapShield™ Wound Retractor

Assembly and Disassembly



## Assembly

1- Prepare the surgical site according to standard procedure.

Make sure the skin is clean and dry.

2- Place the template over the incision site, and mark an appropriate length incision line using a sterile marker.



# Assembly

3- Make an incision along the marked incision line.



# Assembly

4- Insert the “blue ring” of the Lapis LapShield™ into the operative site.



# Assembly

5- Gently grasp the retaining “green ring” of the Lagis LapShield™ at the 10 o’clock and 2 o’clock position and roll inward repeatedly until ultimate retraction is obtained.



# Assembly

6- Attach the retractor attachment ring onto the retaining green ring to make the wound retractor circular and completely self-retaining.





# Disassembly



Grasp the retaining green ring and make an outward roll to loosen the wound retractor.



Retrieve the Lapis LapShield™ Wound Retractor by simply removing the blue ring from the peritoneal cavity.