Anchor™ Tissue Retrieval System™ by CONMED

DEVICE DESCRIPTION

The Anchor™ Tissue Retrieval System™ by CONMED is a sterile disposable pouch used with a dedicated introducer for the encapture and removal of an organ or tissue from the body cavity during laparoscopic or minimally invasive surgery. The Anchor™ Tissue Retrieval System™ by CONMED has been designed in several configurations for a wide variety of procedures for safe specimen retrieval. This product is deployable more than once on the same patient during the same procedure.

INDICATION FOR USE

The Anchor™ Tissue Retrieval System™ by CONMED is a sterile disposable pouch used with a dedicated introducer for the encapture and removal of an organ or tissue from the body cavity during laparoscopic surgery.

CONTRAINDICATIONS

The range of Anchor™ Tissue Retrieval System™ by CONMED are contraindicated when, in the judgement of the doctor or physicians, the use of such a system would not be in the best interest of the patient.

WARNINGS

- Only Physicians trained in the techniques of laparoscopic or minimally invasive surgery and the use of retrieval bags to remove tissue should use the product.
- The Anchor™ Tissue Retrieval Systems™ by CONMED are for single patient use only and should not be re-sterilized. The ability to effectively clean and reuse this single use device has not been established and subsequent re-use may adversely affect the performance, safety and/or stability of the device.
- The use of morcellators within the Anchor™ Tissue Retrieval System™ by CONMED has not been evaluated. The Anchor™ Tissue Retrieval System™ by CONMED is not recommended for use with power morcellators.

PRECAUTIONS

- Care should be taken when using sharp and electro-surgical devices.
- Note the size of the trocar cannula when removing a specimen through the trocar cannula and seal system. If required, enlarge the port site to aid removal of the Anchor™ Tissue Retrieval System™ by CONMED.

Directions for Use

Step 1

Pull handle back

Step 2

Pull bag into introducer

Step 3

Insert the Anchor™ Tissue Retrieval System™ by CONMED through the trocar cannula or body orifice and push the bag out of introducer

Step 4

Place specimen into the bag

Step 5

If redeployment is not required, proceed to step 6

If redeployment is desired, retract the handle to cinch the bag and then remove the Anchor™ Tissue Retrieval System™ by CONMED through the body cavity orifice. Once outside, push the handle to deploy the bag and remove the specimen from the bag.

Repeat steps 1-4 to redeploy Anchor™ Tissue Retrieval System™ by CONMED

Step 6

While pushing button, advance handle to release the latch


Step 7

Pull handle back from introducer

Step 8

Remove handle from introducer, cinch drawstring and inspect thoroughly and see that stainless steel arms, toggle and spring are attached, before removing the specimen from the body cavity. After removing specimen, dispose of Anchor™ Tissue Retrieval System™ by CONMED per facility protocol.

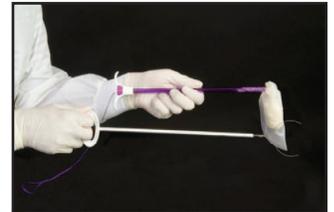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CONMED Corporation
MDSS GmbH